

Universal Precaution Compliance Kit Directions

Kit Numbers: (17100, 17102)

Kit Contents:

- Pair of Vinyl Gloves
- Protective Apron
- Combo Mask/Safety Shield
- Red Z[®] Solidifier (10g. Pouch)
- Scoop/Scraper
- SaniZide Pro[®] Germicidal Wipe (Dimensions: 8" x 8")
- Red Bio-hazard Waste Bag (Dimensions: 24" x 24")
- Twist Tie
- p.a.w.s.[®] Antimicrobial Hand Wipes (Quantity: 1 / Dimensions: 5" x 8")
- Identification Tag
- Instructions

Directions:

1. Put on protective apron, safety mask and gloves.
2. Sprinkle Red Z[®] over spilled area. Allow liquid to congeal for safer handling and transport.
3. Remove gelled material with scoop and scraper. Carefully place in red bio-hazard bag.
4. Use SaniZide Pro[®] Disinfectant Wipe to clean contaminated surface.
5. After procedure is completed remove gloves, safety mask and apron... Place all contaminated materials in the red bio-hazard bag.
6. Seal bag with supplied twist tie. Fill out identification tag and attach. Dispose of bag as required by local, state and federal regulations for infected solid waste.
7. Wipe hands with p.a.w.s.[®] Antimicrobial Hand Wipe. Wash with soap and water as soon as possible.