



Veterinary Use

Safetec SaniZide Pro 1 (1-Minute Surface Disinfectant)

Protect animals from harmful germs and make patient turnover times even faster with Safetec's SaniZide Pro 1 Surface Disinfectant. This ready to use, alcohol based formula combines cleaning and disinfecting in one simple step with true, one-minute kill claims that are effective at killing over 48 microorganisms including **Canine Parvovirus, Canine Distemper virus, Enterococcus, Streptococcus, and Trichophyton (Ringworm).**

SaniZide Pro 1 is non-abrasive and safe for repeated use on hard, non-porous surfaces associated with non-food contact countertops, exam tables, cages, kennels, animal equipment, devices found in animal housing facilities, and more.

- **Removes dirt, grime, blood, urine, and fecal matter**
- **Reduces odor-causing bacteria**
- **99.9% effective in killing microorganisms and germs**



Item#	Description	Case Qty.	Case Wt.
35910	SaniZide Pro 1 Spray (32 oz.)	6/case	14 lbs
35926	SaniZide Pro 1 Wipes (50 ct. box)*	6/case	12 lbs
35923	SaniZide Pro 1 Wipes (150 ct. canister)**	12/case	23 lbs

*Wipe Size: 7.5" x 8"

**Wipe Size: 6" x 6.75"



For use in:

- Animal Hospitals
- Animal Life Science Laboratories
- Animal Transportation Vehicles
- Breeding Establishments
- Grooming Establishments
- Kennels
- Pet Shops
- Tack Shops
- Veterinary Clinics
- Veterinary Examination Rooms
- Veterinary Operating Rooms
- Veterinary Waiting Rooms
- Veterinary X-Ray Rooms
- Zoos



Safetec of America, Inc.
1-800-456-7077 | safetec.com



Directions For Use

Veterinary Clinics, Animal Life Science Laboratories, Pet Shops: Remove all animals and feed from premises, vehicles, and enclosures. Remove all litter and manure from floors, walls, and surfaces of facilities and fixtures occupied or traversed by animals. Empty all troughs, racks and other feeding and watering appliances.

Thoroughly clean all surfaces with soap or detergent and rinse with water. Using either SaniZide Pro 1 Spray or Wipes, wet surfaces for 1 minute. Immerse all handling and restraining equipment as well as forks, shovels, and scrapers used for removing litter and manure.

Ventilate buildings, vehicles, and other closed spaces. Do not house animals or employ equipment until treatment has been absorbed, set, or dried. Thoroughly scrub treated feed troughs, racks and other feeding and watering appliances with soap or detergent, then rinse with potable water before reuse.

Nonrefillable container. Do not reuse or refill this container. Offer for recycling if available, or discard empty container in trash.

SaniZide Pro 1 Efficacy Information (Veterinary)

†Enveloped/lipophilic type virus ††Non-enveloped/hydrophilic type virus

Test Organism

†Avian Influenza A Virus.....	1-minute
Bordetella bronchiseptica	1-minute
Bovine Diarrhea Virus.....	1 minute
†Canine distemper virus	1-minute
††Canine parvovirus	1-minute
Duck Hepatitis B Virus.....	1-minute
Enterobacter aerogenes or Enterobacter cloacae	1-minute
††Enterovirus EV-D68	1-minute
†Feline leukemia virus	1-minute
Streptococcus pneumoniae	1-minute
Streptococcus pyogenes	1-minute
Trichophyton mentagrophytes (<i>Ringworm, Athlete's Foot, Jock Itch</i>)	1-minute

Contact Time

May be used on hard, non-porous surfaces in:

Animal Housing/Holding Areas: Animal Hospitals, Animal Life Science Laboratories, Animal Transportation Vehicles, Breeding Establishments, Grooming Establishments, Kennels, Pet Shops, Tack Shops, Veterinary Clinics, Veterinary Examination Rooms, Veterinary Operating Rooms, Veterinary Waiting Rooms, Veterinary X-Ray Rooms, Zoos.



Safetec of America, Inc.
1-800-456-7077 | safetec.com